Qufora® IrriSedo Balloon

EN Qufora® IrriSedo Balloon Instructions for Use

SV Qufora® IrriSedo Balloon Bruksanvisning

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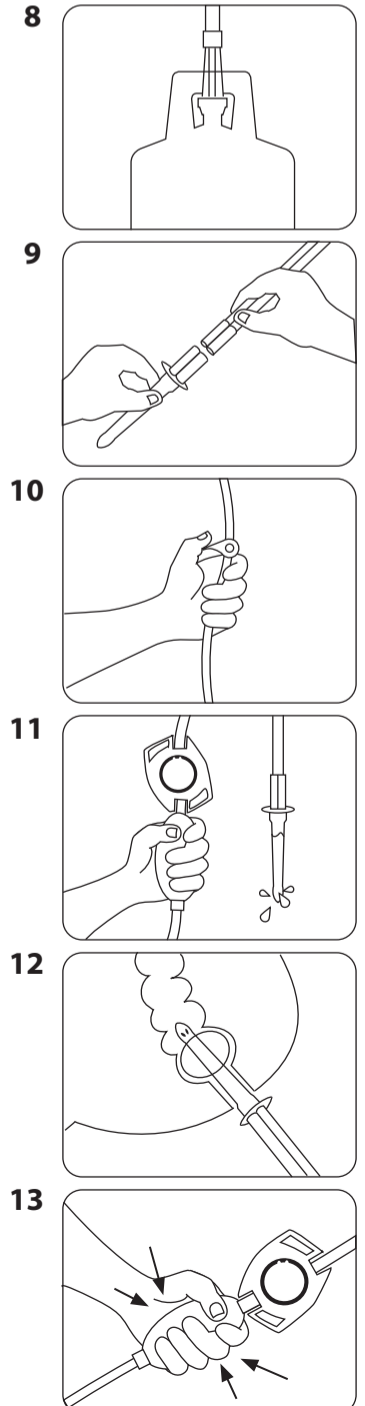
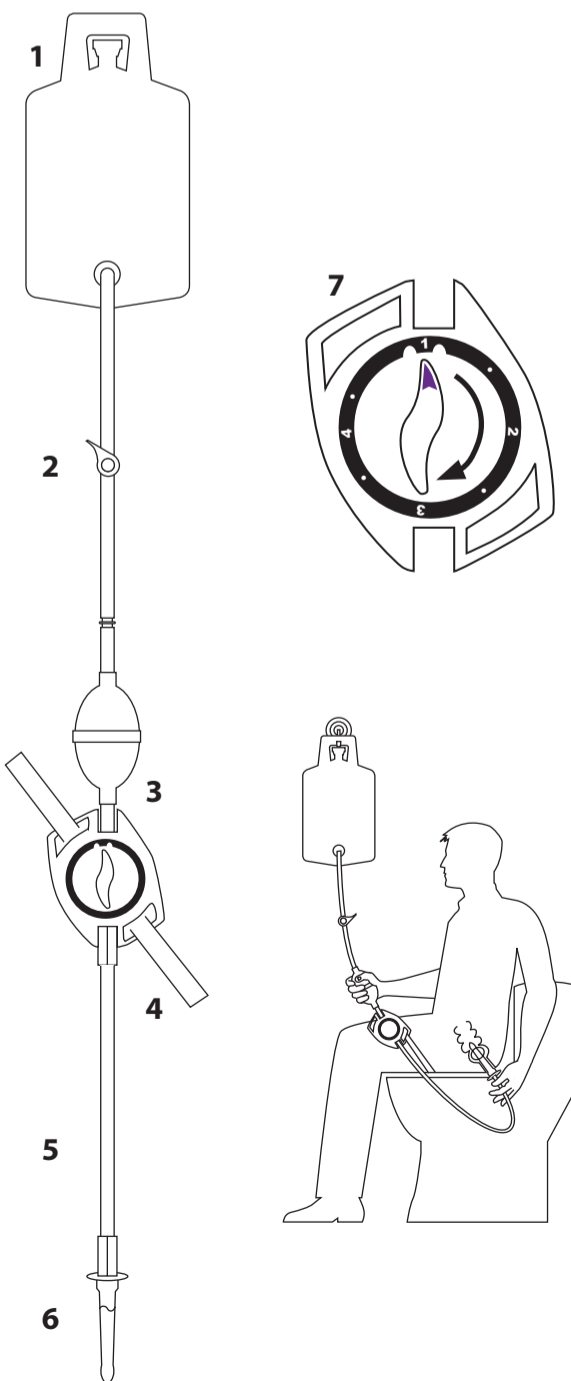
Qufora A/S
Gydevang 28-30
3450 Allerød
Denmark

Tel: +45 88 300 592
Fax: +45 88 300 593
info@qufora.com
www.qufora.com

Qufora® is a registered trademark of Qufora A/S. All rights reserved Qufora A/S, 3450 Allerød, Denmark.

English

Qufora® IrriSedo Irrigation System

Using the Qufora® IrriSedo Irrigation System makes it possible to perform rectal irrigation.

Please read both pages of this instruction before using the Qufora® IrriSedo Irrigation System.

Application

The Qufora® IrriSedo Irrigation System is used to prevent chronic faecal incontinence and/or constipation or for those who spend a long time on bowel management. It should be used regularly - usually daily or every other day to provide the best effect.

Before starting rectal irrigation

Use only after assessment, approval and full instruction by a doctor/health care professional who is specialised in this procedure. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Indications for using rectal irrigation

Individuals with the following condition and symptoms may benefit:

- Neurogenic bowel dysfunction, e.g. spinal cord injury, spina bifida and multiple sclerosis.
- Chronic constipation.
- Chronic faecal incontinence.

Within these groups some individuals may require support of a healthcare professional to perform the procedure.

Irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis.
- Colorectal cancer pre-surgical removal.
- Acute inflammatory bowel disease.
- Acute diverticulitis.
- Within 3 months of anal or colorectal surgery.
- Within 4 weeks of endoscopic polypectomy.
- Ischaemic colitis.

Since the list is not exhaustive a doctor/healthcare professional should always consider individual patient factors as well.

Use rectal irrigation only after careful discussion with a relevant medical practitioner under the following circumstances (relative contraindications):

- Pregnancy or planned pregnancy or breast feeding.
- Children under 3 years of age, considering the size and shape of the device delivering the water. For child irrigation the approval of a qualified medical practitioner specialised in paediatric care is required.
- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis).
- Active perianal sepsis (fistula or abscess).
- Rectal or colonic surgery within the past 6 months.
- Diarrhoea of unknown aetiology.
- Faecal impaction/rectal constipation.
- Severe autonomic dysreflexia.
- Severe diverticulosis or diverticular abscess.
- Abdominal or pelvic radiotherapy.
- Known diverticular disease.
- Long term steroid therapy.
- Anti-coagulant therapy.
- Low blood sodium.
- Pelvic malignancy.
- Colonic biopsy within the past 3 months.
- Use of rectal medications for other diseases which may be diluted by irrigation.
- Congestive cardiac failure.

Warning

Perforation of the bowel wall is extremely rare. However, it is a serious and potentially lethal complication of rectal irrigation. Should this occur, immediate medical attention is required.

Seek medical attention if the following is experienced during or after rectal irrigation:

- Sustained or severe bleeding from the rectum.
- Sustained or severe abdominal or back pain, with or without fever.

If the single use items of the product are reused, there is a risk of unwanted effects that can influence the function of the product and the health of the patient, the caregivers and third persons.

Please observe

Rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light passing bleeding from the rectum.

By following the recommended water volumes and water temperatures the bowel electrolyte balance is unlikely to be affected.

Recommended usage time (when used routinely every other day)

The balloon catheter is a single use item.

The water bag can be used up to 1 month.

The pump and control unit can be used up to 6 months.

Should discolouration develop on any part of the system within the recommended usage time, the part should be replaced.

Liability

Qufora A/S accepts no liability for any injury or other loss that may arise if this product is used in any way contrary to the recommendations of Qufora A/S.

Handling and storage

Qufora® IrriSedo Irrigation System should be stored at temperatures between 0 and 30° C. Protect the product from direct sunlight and do not expose to heavy pressure.

The product should be kept out of reach of children.

Further information about this product can be found on www.qufora.com.

Distributor United Kingdom

MacGregor Healthcare Ltd
Unit 2A, Euro House
Satellite Park, Macmerry
East Lothian
EH33 1RW

Distributor Ireland

Unipharm Medical,
4045 Kingswood Road,
Citywest Business Park,
Co. Dublin
D24 V06K

How to use the system

The system comprises:

- (Fig. 1) Water bag that holds up to 1.5 litres with a tube fitted with a valve
- (Fig. 2) Valve
- (Fig. 3) Pump and control unit
- (Fig. 4) Velcro strap
- (Fig. 5) Double lumen tube
- (Fig. 6) Coated rectal catheter with a silicone balloon
- (Fig. 7) Control unit

The numbers on the control unit indicate the following functions:

- Position 1** - Fills the tubes with water/dries the system after use.
- Position 2** - Fills the balloon with water to retain the balloon in the rectum.
- Position 3** - Water runs through the catheter into the rectum.
- Position 4** - Water evacuates the balloon to allow the catheter to be removed from the rectum.

Between the numbered positions, 4 “stop positions” are indicated by white dots.

Use irrigation only after assessment and approval by a qualified health care professional.

To use the system, please follow the instructions:

- 1 Fill the water bag (Fig. 8) with warm (36 – 38 °C) tap water (only fill the bag up to the amount of water you need for your irrigation) and fit the lid. Place the water bag no higher than 50 cm above the toilet seat. The length of the tube from water bag to the valve is 50 cm. Connect the water bag to the pump and regulator.
- 2 Push the catheter onto one end of the double lumen tube (Fig. 9) and fit the other end of the tube to the control unit. Use the Velcro strap to attach the control unit to the thigh. Turn the control unit to **position 1**.
- 3 Hold the pump with the control unit pointing upwards and open the valve (Fig. 10). Compress the pump slowly to fill the pump and tubes with water (Fig. 11) until water runs out of the catheter. Turn the control unit to **position 2**, ready to fill the balloon. Dip the catheter in water to activate the lubrication
- 4 Seated on the toilet, carefully insert the catheter into the rectum (Fig. 12).
- 5 The balloon is filled by compressing the pump completely. If the catheter needs to be further secured, compress the pump again.
- 6 Turn the control unit to **position 3**. Water can now be pumped into the rectum (Fig. 13). Allow approximately 300 to 500 ml (less for children as specified by the healthcare professional) to enter the rectum and set the control unit between position 3 and 4. The amount of water can be monitored on the bag scale. Use max. 1000 ml. per irrigation. If any discomfort is felt, do not instil more water into the bowel.
- 7 When the water has been instilled in the bowel, turn the control unit to **position 4**. The water from the balloon will flow into the rectum and the catheter can be removed.
- 8 Water and faecal matter will flow into the toilet.
- 9 The procedure can be repeated by repeating step 4 – 8.
- 10 After irrigation wash the outside of the water bag, the control unit and double lumen tube with mild soap. Disassemble the double lumen tube from the regulator. Empty the water bag and hang the double lumen tube and control unit individually to dry with the control unit in position 1 and the lid off the water bag.
- 11 Do not re-use the balloon catheter. It should be disposed of by putting it into a disposal bag before putting it in the household waste. Do not flush down the toilet. Wash your hands to complete the procedure.

To gain faecal continence or to resolve constipation, irrigation should be used routinely daily or every other day.

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Quforal A/S