Qufora® IrriSedo Cone

EN Qufora® IrriSedo Cone **Instructions for Use**
SV Qufora® IrriSedo Cone **Bruksanvisning**
DE Qufora® IrriSedo Cone **Gebrauchsanweisung**
DA Qufora® IrriSedo Cone **Brugsanvisning**
NL Qufora® IrriSedo Cone **Gebruiksaanwijzing**
FI Qufora® IrriSedo Cone **Käyttöohjeet**
NO Qufora® IrriSedo Cone **Bruksanvisning**
ES Qufora® IrriSedo Cone **Instrucciones de uso**
EL Σύστημα κώνου Qufora® IrriSedo **Οδηγίες χρήσης**
FR Qufora® IrriSedo Cone **Mode d'emploi**
IT Qufora® IrriSedo Cone **Istruzioni d'uso**
HE Qufora® IrriSedo **קונוס למערכת שימוש**
AR Qufora® IrriSedo Cone **ناهج مداخلت سالا تامي لعت**

3396-05_180122








Qufora A/S
Gydevang 28-30
3450 Allerød
Denmark

Tel: +45 88 300 592
Fax: +45 88 300 593
info@qufora.com
www.qufora.com

Qufora® is a registered trademark of Qufora A/S. All rights reserved Qufora A/S, 3450 Allerød, Denmark.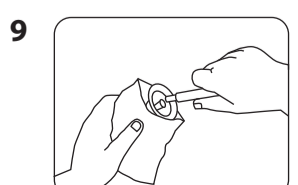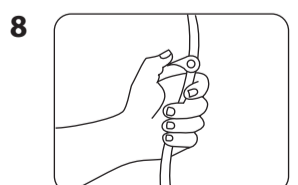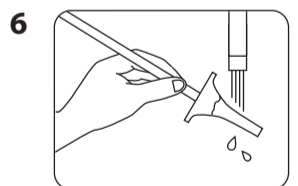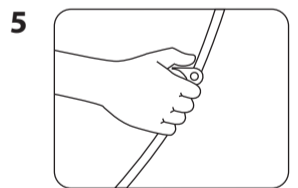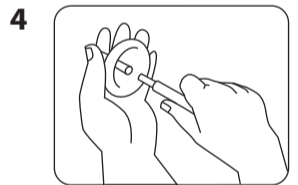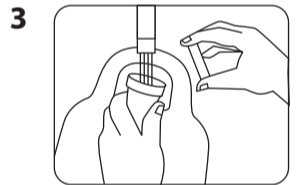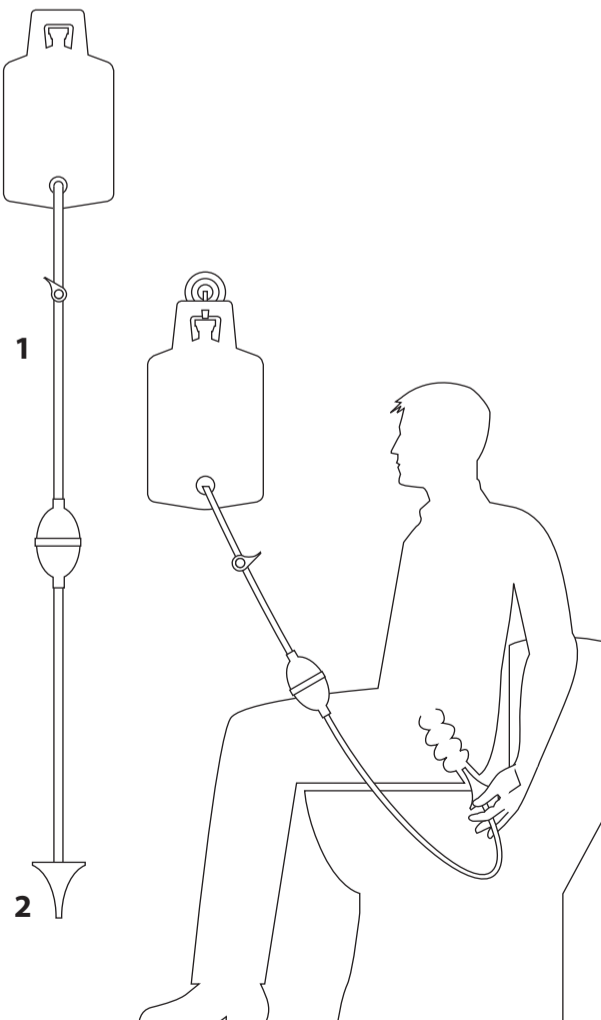

English

Qufora® IrriSedo Cone System

Using the Qufora® IrriSedo Irrigation System makes it possible to perform rectal irrigation. Please read both pages of this instruction before using the Qufora® IrriSedo Irrigation System.

Application

The Qufora® IrriSedo Irrigation System is used to prevent chronic faecal incontinence and/or constipation or for those who spend a long time on bowel management. It should be used regularly - usually daily or every other day to provide the best effect.

Before starting rectal irrigation

Use only after assessment, approval and full instruction by a healthcare professional who is specialised in this procedure. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Indications for using rectal irrigation

Individuals with the following conditions and symptoms may benefit:

- Neurogenic bowel dysfunction, e.g. spinal cord injury, spina bifida and multiple sclerosis.
- Chronic constipation.
- Chronic faecal incontinence.

Within these groups some individuals may require the support of a carer to perform the procedure.

Irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis.
- Colorectal cancer pre surgical removal.
- Acute inflammatory bowel disease.
- Acute diverticulitis.
- Within 3 months of anal or colorectal surgery.
- Within 4 weeks of endoscopic polypectomy.
- Ischaemic colitis.

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Use rectal irrigation only after careful discussion with relevant medical practitioner under the following circumstances (relative contraindications):

- Pregnancy or planned pregnancy or breast feeding.
- Children under 3 years of age, considering the size and shape of the device delivering the water. For children the approval should be from a healthcare professional specialised in paediatric care.
- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis).
- Active perianal sepsis (fistula or abscess).
- Rectal or colonic surgery within the past 6 months.
- Diarrhoea of unknown aetiology.
- Faecal impaction/ rectal constipation.
- Severe autonomic dysreflexia.
- Severe diverticulosis or diverticular abscess.
- Abdominal or pelvic radiotherapy.
- Known diverticular disease.
- Long term steroid therapy.
- Anti-coagulant therapy.
- Low blood sodium.
- Pelvic malignancy.
- Colonic biopsy within the past 3 months.
- Use of rectal medications for other diseases which may be diluted by irrigation.
- Congestive cardiac failure.

Warning

Perforation of the bowel wall is extremely rare. However it is a serious and potentially lethal complication of rectal irrigation. Should this occur, immediate medical attention is required.

Seek medical attention if the following is experienced during or after rectal irrigation:

- Sustained or severe bleeding from the rectum.
- Sustained or severe abdominal or back pain, with or without fever.

If the single use items of the product are reused, there is a risk of unwanted effects that can influence the function of the product and the health of the patient, the carers and third persons.

Please observe

Rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum.

By following the recommended water volumes and water temperatures the bowel electrolyte balance is unlikely to be affected.

Recommended usage time (when used routinely every other day)

The water bag and pump can be used up to 1 month. Should discolouration develop within the recommended usage time, this item should be replaced.

The cone is a single use item.

Liability

Qufora A/S accepts no liability for any injury or other loss that may arise if this product is used in any way contrary to the recommendations of Qufora A/S.

Handling and storage

Qufora® IrriSedo Irrigation System should be stored at temperatures between 0 and 30° C.

Protect the product from direct sunlight and do not expose to heavy pressure.

The product should be kept out of reach of children.

Further information about this product can be found on www.qufora.com.

Distributor United Kingdom

MacGregor Healthcare Ltd
Unit 2A, Euro House
Satellite Park, Macmerry
East Lothian
EH33 1RW

Distributor Ireland

Uniphar Medical,
4045 Kingswood Road,
Citywest Business Park,
Co. Dublin
D24 VO6K

How to use the system

The system comprises:

(fig. 1) A water bag that holds up to 1.5 litres of water fitted with a valve, a pump and a connector.
(fig. 2) A soft cone coated with a hydrophilic lubricant that is activated by water.

- 1 Use irrigation only after assessment and approval by a qualified healthcare professional.

To use the system please follow the instructions:

- 2 Make sure the valve is closed. Fill the water bag with warm (36°C - 38°C) tap water (only fill the bag with 100 ml more than you need for your irrigation) (fig.3). Hang the water bag no higher than 50 cm above the level of the toilet seat. The length of the tube from the water bag to the valve is 50 cm.

- 3 Connect the water bag tube connector to the cone (fig.4). The sticky label on the cone pouch can be attached to a surface within convenient reach. The cone pouch is ready to receive the cone after use.

- 4 Open the valve (fig. 5) to fill the tube and pump with water and close it again.

- 5 To activate the lubricant simply bring the cone into contact with water (fig. 6).

- 6 Seated on the toilet, carefully guide the cone into the rectum (fig. 7).

- 7 With the cone held in position with one hand, open the valve. Start by opening the valve about half way (fig. 8). If the water bag is placed lower than the toilet seat, open the valve completely and pump the water into the rectum.

- 8 Allow approximately 300 – 500 ml. of water (less for children – as specified by your healthcare professional) to enter the rectum and close the valve. The amount of water can be monitored on the water bag scale. **Use max. 1000 ml. per irrigation.** If any discomfort is felt, close the valve immediately.

- 9 Remove the cone. Water and faecal matter will flow into the toilet.

- 10 If necessary, the procedure can be repeated by repeating steps 5 to 8.

Place the soiled cone into the cone pouch (fig. 9). Hold the cone through the cone pouch to twist and pull the cone off the tube. Fold the open end of the cone pouch twice and use the sticky label to close it. Throw it away in your household waste.

Clean the pump with mild soap. Empty the water bag and hang it to dry with the lid off.

Wash your hands to complete the procedure.

To gain faecal continence or to resolve constipation, irrigation should be used routinely daily or every other day.

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