Qufora® IrriSedo Balloon

EN	Instructions for Use
SV	Bruksanvisning
DE	Gebrauchsanweisung
DA	Brugsanvisning
NL	Gebruiksaanwijzing
FI	Käyttöohjeet
NO	Bruksanvisning
ES	Instrucciones de uso
FR	Mode d'emploi
IT	Istruzioni d'uso
HE	שומיש תוארה
AR	مادخت سالات امي لع ت
EL	Οδηγίες χρήσης

3394-09_250325


www.qufora.com

Qufora A/S
Gydevang 28-30
3450 Allerød
Denmark

Tel: +45 88 300 592
info@qufora.com
www.qufora.com

Qufora® is a registered trademark of Qufora A/S. © 03/2025. All rights reserved Qufora A/S, 3450 Allerød, Denmark.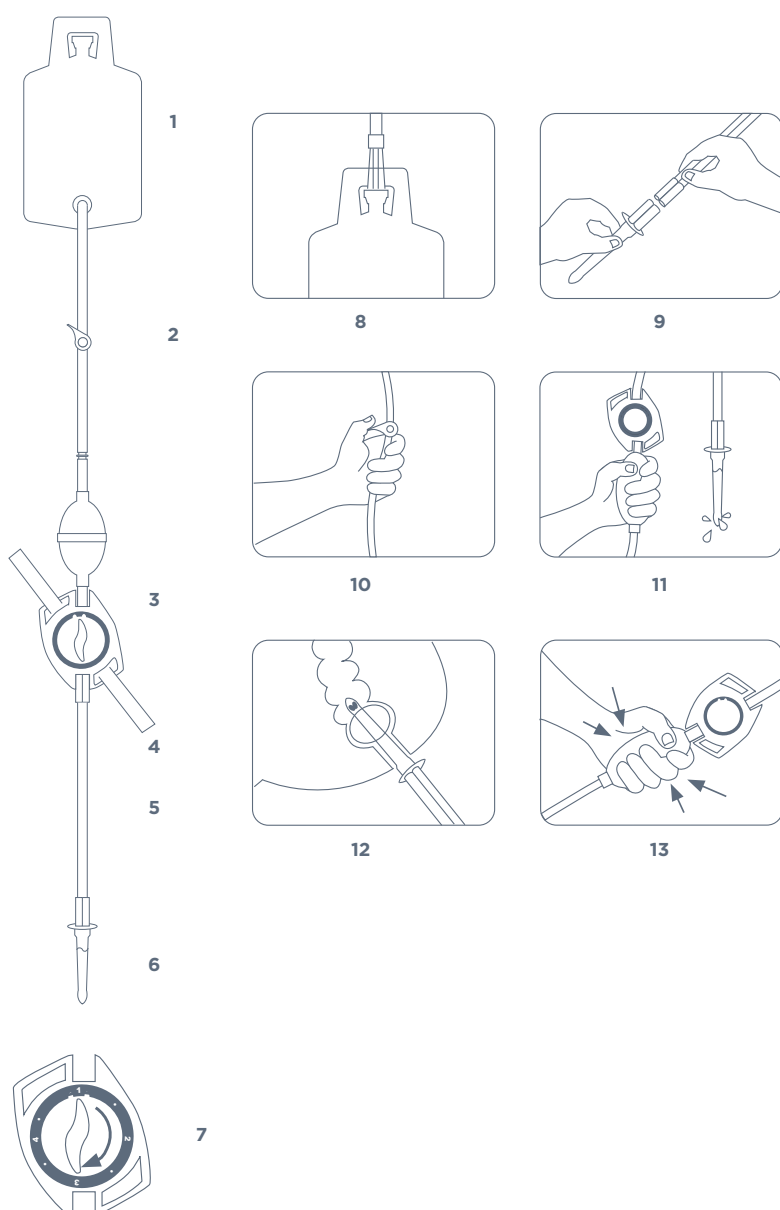

Instructions for use

Qufora IrriSedo Balloon

Using the Qufora IrriSedo Balloon makes it possible to perform transanal/rectal irrigation. Please read both pages of this instructions for use before using the Qufora IrriSedo Balloon.

Indications for using transanal/rectal irrigation

The Qufora IrriSedo Balloon is intended for adults and children 3 years and above to help manage faecal incontinence and/or constipation conditions.
For example:

- Urge faecal incontinence/urgency
- Constipation (slow transit/idiopathic/opioid induced/IBS-C)
- Neurogenic (spinal cord injury, upper/lower motor neurone/MS/parkinson's/spina bifida/cauda equina)

Within these groups some individuals may require the support of a carer to perform the procedure.

Use as advised by your healthcare professional

For the best results Qufora IrriSedo Balloon should be used regularly as advised by your healthcare professional.

Before starting transanal/rectal irrigation

Use only after assessment and training from a qualified healthcare professional. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Transanal/rectal irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis
- Colorectal cancer/pelvic malignancy pre surgical removal
- Acute inflammatory bowel disease
- Acute diverticulitis
- Within 3 months of anal or colorectal surgery
- Within 4 weeks of endoscopic polypectomy
- Ischaemic colitis

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Please note:

- Transanal/rectal irrigation is not recommended for:
- Pregnancy or planned pregnancy or breastfeeding
 - Children under 3 years of age

Use transanal/rectal irrigation only after careful discussion with a relevant medical practitioner under the following circumstances (relative contraindications):

- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis)
- Active perianal sepsis (fistula or abscess, third or fourth degree haemorrhoids)
- Previous rectal or colonic surgery
- Diarrhoea of unknown aetiology
- Faecal impaction/rectal constipation. If the patient is heavily constipated (faecally impacted) an initial clean-out of the bowel is mandatory before starting up on Qufora IrriSedo Balloon Transanal Irrigation procedure
- Severe autonomic dysreflexia
- Severe diverticulosis or diverticular abscess
- Abdominal or pelvic irradiation
- Long term steroid therapy
- Anti-coagulant therapy
- Low blood sodium
- Previous severe pelvic surgery
- Colonic biopsy within the past 3 months
- Use of rectal medications for other diseases which may be diluted by irrigation
- Congestive cardiac failure

Adverse events and device malfunction

Seek medical attention if the following is experienced during or after transanal/rectal irrigation:

- Sustained or severe bleeding from the rectum
- Sustained or severe abdominal or back pain, with or without fever

Perforation of the bowel wall is extremely rare. However it is a serious and potentially fatal complication of transanal/rectal irrigation. Should this occur, immediate medical attention is required. The incidents should be reported to the competent authority in your country and/or to Qufora A/S at QA@qufora.com . Qufora A/S should also be notified in case of any other device malfunction.

Please observe

Transanal/rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum.

Please contact your healthcare professional for further advice.

Recommended usage time

The water bag and tube can be used up to 15 times. The pump and control unit can be used up to 90 times.

Should discolouration develop on any part of the system within the recommended usage time, the part should be replaced.

The catheters are single-use items.

If the single-use items of the product are reused, there is a risk of unwanted effects that can influence the function of the product and the health of the patient, the carers and third persons.

Please find further information about this product on www.qufora.com.

Distributor/UKRP:
Qufora Ltd,
Euro House,
Satellite Park, Macmerry,
East Lothian,
EH33 1RW

Explaining the Qufora IrriSedo Balloon components

- Fig. 1 Water bag that holds up to 1.5 litres with a tube fitted with a valve
- Fig. 2 Valve
- Fig. 3 Pump and control unit
- Fig. 4 Fixation strap
- Fig. 5 Double lumen tube
- Fig. 6 Coated rectal catheter with a silicone balloon
- Fig. 7 Control unit

The numbers on the control unit indicate the following functions:

- Position 1 Fills the tubes with water/dries the system after use.
- Position 2 Fills the balloon with water to retain the balloon in the rectum.
- Position 3 Water runs through the catheter into the rectum.
- Position 4 Water evacuates the balloon to allow the catheter to be removed from the rectum.
- Dots Between the numbered positions, 4 “stop positions” are indicated by white dots.

To use the system please follow the instructions:

1. Fill the water bag (Fig. 8) with luke warm tap water (36 - 38°C) (you will need more water than you need for your irrigation to prepare your system, approximately 100-300 ml water) and fit the lid. Place the water bag no higher than 50 cm above the toilet seat. The length of the tube from water bag to the valve is 50 cm. Connect the water bag to the pump and regulator.
2. Push the catheter onto one end of the double lumen tube (fig. 9) and fit the other end of the tube to the control unit. Use the fixation strap to attach the control unit to the thigh. Turn the control unit to position 1.
3. Hold the pump with the control unit pointing upwards and open the valve (fig. 10). Compress the pump slowly to fill the pump and tubes with water (fig. 11) until water runs out of the catheter. Turn the control unit to position 2, ready to fill the balloon.
4. Dip the catheter in water to activate the lubrication
5. Seated on the toilet, carefully insert the catheter into the rectum (fig. 12).
6. The balloon is filled by compressing the pump completely. If the catheter needs to be further secured, compress the pump again (fig. 12).
7. Turn the control unit to position 3. Water can now be pumped into the rectum (fig. 13). The amount of water flowing into the bowel can be monitored by using the scale on the water bag. Your healthcare professional will guide you regarding the amount of water to use for your irrigation. A general guide is 500 ml (this will vary for children) however this may be more or less depending on your requirement. If you feel any discomfort, do not instil more water into the bowel. Stop the flow by turning the control unit to the dot between 3 and 4.
8. When the water has been instilled in the bowel, turn the control unit to position 4. The water from the balloon will flow into the rectum and the catheter can be removed.vWater and faecal matter will flow into the toilet.
9. The procedure can be repeated by repeating step 4 - 8.
10. If required, all external surfaces of the system can be cleaned with luke warm water and mild soap. Dry with a lint-free cloth. Disassemble the double lumen tube from the regulator. Empty the water bag and hang the double lumen tube and control unit individually to dry with the control unit in position 1 and the lid off the water bag.
11. Do not re-use the balloon catheter. It should be disposed of by putting it into a disposal bag before putting it in the household waste. Do not flush down the toilet.
12. Wash your hands to complete the procedure.

Please note: Please use Qufora IrriSedo Balloon only as advised by your healthcare professional.

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