Qufora® IrriSedo Cone

EN	Instructions for Use
SV	Bruksanvisning
DE	Gebrauchsanweisung
DA	Brugsanvisning
NL	Gebruiksaanwijzing
NO	Bruksanvisning
ES	Instrucciones de uso
FR	Mode d'emploi
IT	Istruzioni d'uso
HE	שומיש תוארה
AR	مادخت سال ا تامي لعت

3396-08_250325



www.qufora.com

Qufora A/S
Gydevang 28-30
3450 Allerød
Denmark

Tel: +45 88 300 592
info@qufora.com
www.qufora.com

Qufora® is a registered trademark of Qufora A/S. © 03/2025. All rights reserved Qufora A/S, 3450 Allerød, Denmark.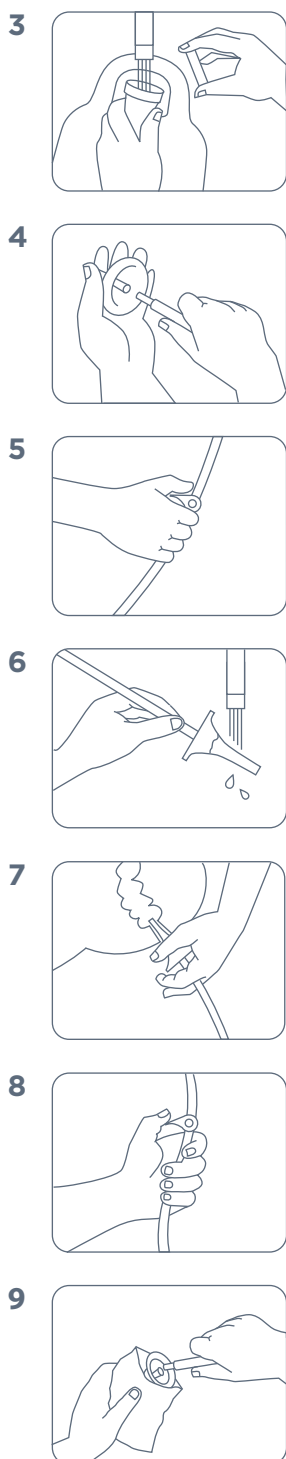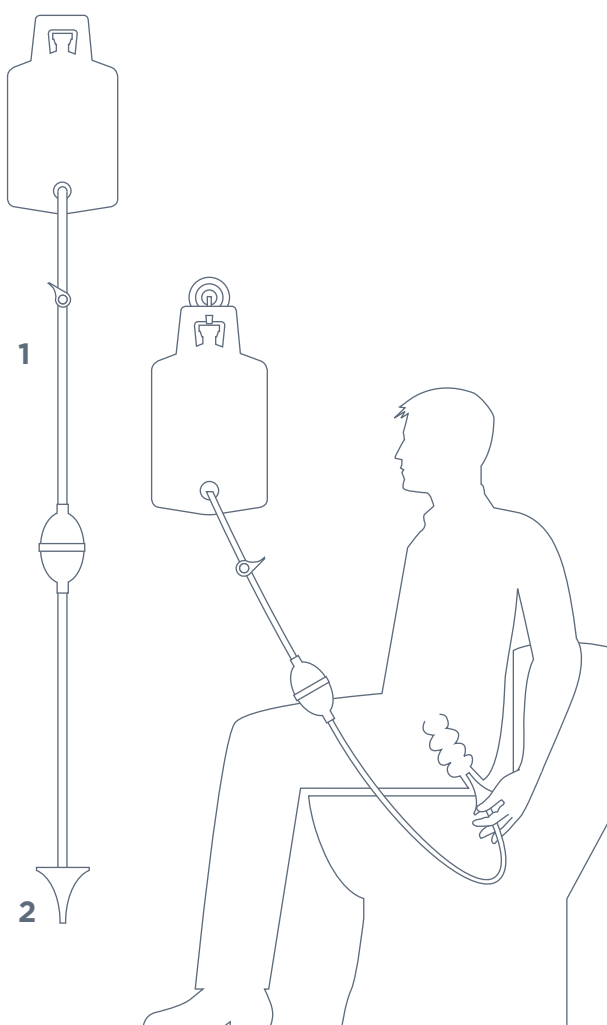

Instructions for use

Qufora IrriSedo Cone

Using the Qufora IrriSedo Cone makes it possible to perform transanal/rectal irrigation. Please read both pages of this instructions for use before using the Qufora IrriSedo Cone.

Indications for using transanal/rectal irrigation

The Qufora IrriSedo Cone is intended for adults and children 3 years and above to help manage faecal incontinence and/or constipation conditions.
For example:

- Low anterior resection syndrome
- Urge faecal incontinence/urgency
- Constipation (slow transit/ idiopathic/ opioid induced/IBS-C)
- Neurogenic (spinal cord injury, upper/lower motor neurone/MS/ parkinson's/spina bifida/cauda equina)

Within these groups some individuals may require the support of a carer to perform the procedure.

Use as advised by your healthcare professional

For the best results Qufora IrriSedo Cone should be used daily or as advised by your healthcare professional.

Before starting transanal/rectal irrigation

Use only after assessment and training from a qualified healthcare professional. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Transanal/rectal irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis
- Colorectal cancer/pelvic malignancy pre surgical removal
- Acute inflammatory bowel disease
- Acute diverticulitis
- Within 3 months of anal or colorectal surgery
- Within 4 weeks of endoscopic polypectomy
- Ischaemic colitis

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Please note:

Transanal/rectal irrigation is not recommended for:

- Pregnancy or planned pregnancy or breastfeeding
- Children under 3 years of age

Use transanal/rectal irrigation only after careful discussion with a relevant medical practitioner under the following circumstances (relative contraindications):

- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis)
- Active perianal sepsis (fistula or abscess, third or fourth degree haemorrhoids)
- Previous rectal or colonic surgery
- Diarrhoea of unknown aetiology
- Faecal impaction/rectal constipation. If the patient is heavily constipated (faecally impacted) an initial clean-out of the bowel is mandatory before starting up on Qufora IrriSedo Cone Transanal Irrigation procedure
- Severe autonomic dysreflexia
- Severe diverticulosis or diverticular abscess
- Abdominal or pelvic irradiation
- Long term steroid therapy
- Anti-coagulant therapy
- Low blood sodium
- Previous severe pelvic surgery
- Colonic biopsy within the past 3 months
- Use of rectal medications for other diseases which may be diluted by irrigation
- Congestive cardiac failure

Adverse events and device malfunction

Seek medical attention if the following is experienced during or after transanal/rectal irrigation:

- Sustained or severe bleeding from the rectum
- Sustained or severe abdominal or back pain, with or without fever

Perforation of the bowel wall is extremely rare. However it is a serious and potentially fatal complication of transanal/rectal irrigation. Should this occur, immediate medical attention is required. The incidents should be reported to the competent authority in your country and/or to Qufora A/S at QA@qufora.com . Qufora A/S should also be notified in case of any other device malfunction.

Please observe

Transanal/rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum.

Please contact your healthcare professional for further advice.

Recommended usage time

The waterbag with pump can be used up to 1 month. Should discolouration develop within the recommended usage time, the water bag with pump should be replaced.

The cone is for single-use only


If the cone is reused, there is a risk of adverse effects to the product and the health of the patient, carers and third persons.

Please find further information about this product on www.qufora.com.

Distributor/UKRP:
Qufora Ltd,
Euro House,
Satellite Park, Macmerrey,
East Lothian,
EH33 1RW


Explaining the Qufora IrriSedo Cone components

Water bag and pump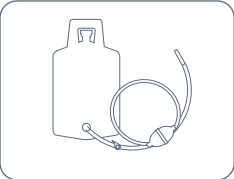

A water bag that holds up to 1.5 litres of water fitted with a valve, a pump and a connector (fig. 1).

Cone

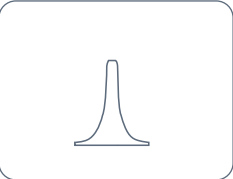A soft cone coated with a hydrophilic lubricant that is activated by water (fig. 2) .

To use the system please follow the instructions:

1. Make sure the valve is closed. Fill the water bag with luke warm tap water (36-38 °C) (you will need more water than you need for your irrigation to prepare your system, approximately 100-300 ml water) and fit the lid (fig. 3). Hang the water bag no higher than 50 cm above the level of the toilet seat. The length of the tube from the water bag to the valve is 50 cm.
2. Connect the water bag tube connector to the cone (fig. 4). The sticky label on the cone pouch can be attached to a surface within convenient reach. The cone pouch is ready to receive the cone after use.
3. Open the valve (fig. 5) to fill the tube and pump with water and close it again.
4. To activate the lubricant simply bring the cone into contact with water (fig. 6).
5. Seated on the toilet, carefully guide the cone into the rectum (fig. 7).
6. With the cone held in position with one hand, open the valve. Start by opening the valve about half way (fig. 8). If the water bag is placed lower than the toilet seat, open the valve completely and pump the water into the rectum.
7. The amount of water flowing into the bowel can be monitored by using the scale on the water bag. Your healthcare professional will guide you regarding the amount of water to use for your irrigation. A general guidance is 500 ml (this will var for children) however this may be more or less depending on your requirement. If you feel any discomfort, do not instil more water into the bowel.
8. Remove the cone. Water and faecal matter will flow into the toilet.
9. If necessary, the procedure can be repeated by repeating steps 5 to 8.
10. Place the soiled cone into the cone pouch (fig. 9). Hold the cone through the cone pouch to twist and pull the cone off the tube. Fold the open end of the cone pouch twice and use the sticky label to close it. Throw it away in your household waste.
11. If required, all external surfaces of the system can be cleaned with luke warm water and mild soap. Dry with a lint-free cloth.
12. Wash your hands to complete the procedure.

Please note: Please use Qufora IrriSedo Cone only as advised by your healthcare professional.

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