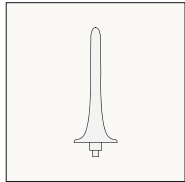
Qufora® IrriSedo Klick

Qufora® IrriSedo Klick

EN	Instructions for Use	4
DE	Gebrauchsanweisung	10
FR	Mode d'emploi	16
IT	Istruzioni per l'uso	22
ES	Instrucciones de uso	28
DA	Brugervejledning	34
SV	Bruksanvisning	40
NO	Bruksanvisning	46
FI	Käyttöohjeet	52
AR	لامعت سالا تامي لعت	58
HE	הוראות שימוש	64


Explaining the system

System components


Rectal catheter
with a hydrophilic coating
and a water inflated
balloon

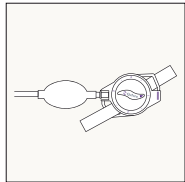- the catheter is single-use


Pump and control unit
with fixation strap

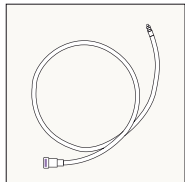- can be used 90 times

Click tube

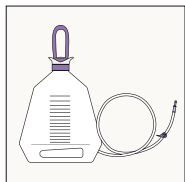
- can be used up to 1 month
(15 times for irrigation every
second day and 30 times
for irrigation every day)

Water bag - 1500 ml with a
tube and a valve


- can be used up to 1 month
(15 times for irrigation every
second day and 30 times
for irrigation every day)

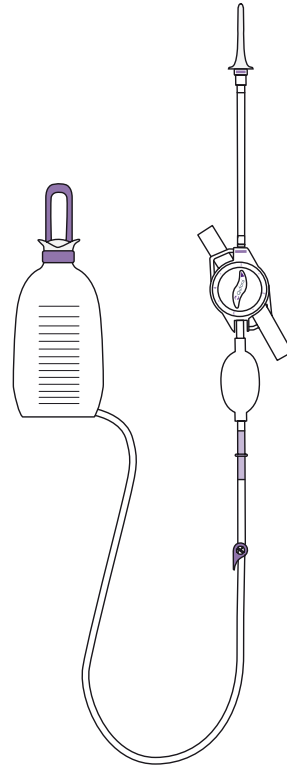System when assembled


Preparing the system for use

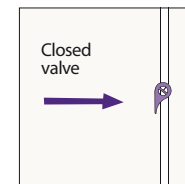
1. Wash and dry your hands.
2. Before you start, make sure the valve on the water bag tube is closed - lever is parallel to the tube.
3. Hold onto the neck of the water bag and fill the water bag with luke warm tap water (36-38° C) by holding it under the tap with the lid open (you will need more water than you need for your irrigation to prepare your system, approximately 100-300 ml water).
4. Once the water bag is filled with the required amount of water, continue to hold onto the neck of the water bag and press down on the flip lid until it is closed.
5. Connect the water bag tube to the pump.
6. Connect the click tube to the control unit - when you hear a click it is connected.
7. The water bag can be placed anywhere between floor and shoulder height from your sitting position. 7a) It can be placed on the floor or, 7b) hanging by the handle.


2

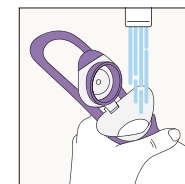
3

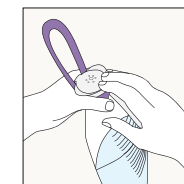
4

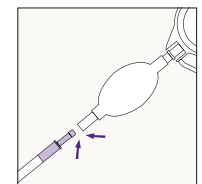
5

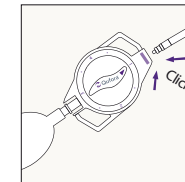6

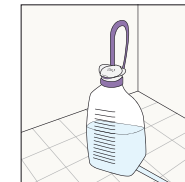
7a

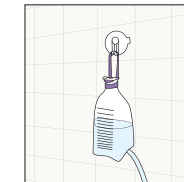
7b

Filling the system with water

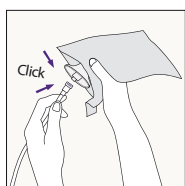
8. Open the catheter pouch at the perforated line. Whilst the catheter is still in the pouch connect the click tube to the catheter - when you hear a click, it is connected. Take the catheter out without touching the hydrophilic coating.
9. Attach the empty pouch to a surface using the sticky area on the back of the pouch.
10. Turn the control unit to position 1.
11. Hold the pump with the control unit pointing upwards (11a) and open the valve on the water bag tube (11b). Squeeze the pump until it is full of water so that no air is left and water comes out of the catheter (11c).
12. Pause the system by turning the control unit to the dot position between 1 and 2.
13. Hold the catheter under the tap or dip it in water up to its base for a few seconds to activate the hydrophilic coating. The system is now ready for use.


8

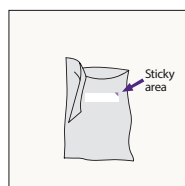
9

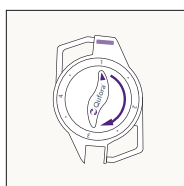
10

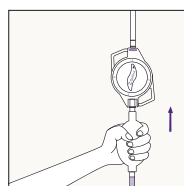11a

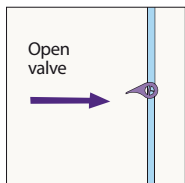
11b

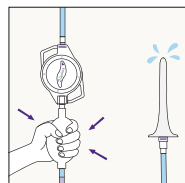11c

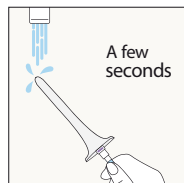
12


13

Using Qufora IrriSedo Klick

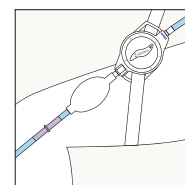
14. Sit on or over the toilet. If required, fasten the control unit to your thigh or wheelchair, using the fixation straps.
15. Carefully insert the catheter into the rectum, supporting the catheter at its base.
16. Turn the control unit clockwise to position 2. Squeeze the pump two to three times (maximum of five complete pumps) to inflate the balloon with water (16a) until the catheter holds in the rectum by itself (16b). Your healthcare professional will advise you on this.
17. Turn the control unit clockwise to position 3. Water can now be instilled into the bowel. If the water bag is hanging, you can either pump the water into the bowel or you can let the water flow into the bowel using gravity. If the water bag is placed lower than the toilet seat, squeeze the pump to instil water into the rectum. If you need to stop the flow turn the control unit to the dot between 3 and 4. To resume irrigation, turn the control unit back to position 3.
18. The amount of water flowing into the bowel can be monitored by using the scale on the water bag. Your healthcare professional will guide you regarding the amount of water to use for your irrigation. A general guide is 500 ml (this will vary for children) however this may be more or less depending on your requirement. If you feel any discomfort, do not instil more water into the bowel. Stop the flow by turning the control unit to the dot between 3 and 4.
19. When you have instilled the required amount of water into the bowel, turn the control unit clockwise to position 4. The balloon will now deflate and the catheter can be carefully removed from the rectum. Water and faecal matter will now be expelled. If this does not happen immediately, try coughing or leaning forward.
20. Close the valve on the water bag tube (lever is parallel to the tube).


14

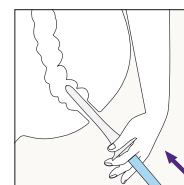
15

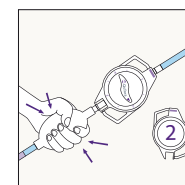
16a

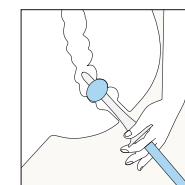
16b

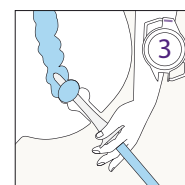
17

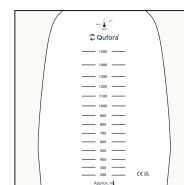18

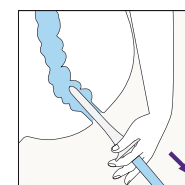
19

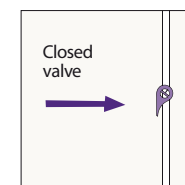
20

Emptying and disassembling the system

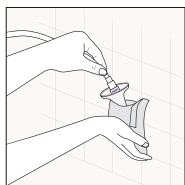
- Click-off the used catheter into the grey pouch by pressing the purple button. Hold the tube upwards to prevent water spillage. The pouch can be folded and sealed using the sticky area, then disposed of in household waste.
- Place the system in the sink for emptying, separating and cleaning.
- Separate the parts by clicking and pulling the connectors apart.
- Empty the water from the system. To empty the control unit and the pump, turn the control unit to position 1 and let the water run out through the control unit (24a). To empty the water bag, open the valve on the water bag and let the water run through the tube (24b) or open the lid on the water bag and empty the bag through the opening (24c).


21

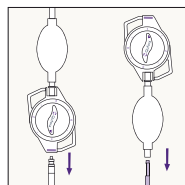23

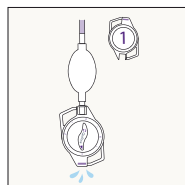24a

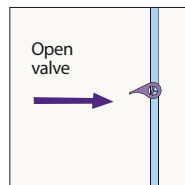24b

24c

Cleaning and storing after use

- If required, all external surfaces of the system can be cleaned with luke warm water and mild soap. Dry with a lint-free cloth.
- Hang the system to dry with the water bag lid and the valve on the water bag tube open and leave the control unit in position 1.
- Wash and dry your hands.

Instructions for use

Qufora IrriSedo Klick

Using the Qufora IrriSedo Klick makes it possible to perform transanal/rectal irrigation. Please read both pages of this instructions for use before using the Qufora IrriSedo Klick.

Indications for using transanal/rectal irrigation

The Qufora IrriSedo Klick is intended for adults and children 3 years and above to help manage faecal incontinence and/or constipation conditions.

For example:

- Urge faecal incontinence/urgency
- Constipation (slow transit/idiopathic/opioid induced/IBS-C)
- Neurogenic (spinal cord injury, upper/lower motor neurone/MS/parkinson's/spina bifida/cauda equina)

Within these groups some individuals may require the support of a carer to perform the procedure.

Use as advised by your healthcare professional

For the best results Qufora IrriSedo Klick should be used regularly as advised by your healthcare professional.

Before starting transanal/rectal irrigation

Use only after assessment and training from a qualified healthcare professional. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Transanal/rectal irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis
- Colorectal cancer/pelvic malignancy pre surgical removal
- Acute inflammatory bowel disease
- Acute diverticulitis
- Within 3 months of anal or colorectal surgery
- Within 4 weeks of endoscopic polypectomy
- Ischaemic colitis

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Please note:

Transanal/rectal irrigation is not recommended for:

- Pregnancy or planned pregnancy or breastfeeding
- Children under 3 years of age

Use transanal/rectal irrigation only after careful discussion with a relevant medical practitioner under the following circumstances (relative contraindications):

- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis)
- Active perianal sepsis (fistula or abscess, third or fourth degree haemorrhoids)
- Previous rectal or colonic surgery
- Diarrhoea of unknown aetiology
- Faecal impaction/rectal constipation. If the patient is heavily constipated (faecally impacted) an initial clean-out of the bowel is mandatory before starting up on Qufora IrriSedo Klick Transanal Irrigation procedure.
- Severe autonomic dysreflexia
- Severe diverticulosis or diverticular abscess
- Abdominal or pelvic irradiation
- Long term steroid therapy
- Anti-coagulant therapy
- Low blood sodium
- Previous severe pelvic surgery
- Colonic biopsy within the past 3 months
- Use of rectal medications for other diseases which may be diluted by irrigation
- Congestive cardiac failure

Adverse events and device malfunction

Seek medical attention if the following is experienced during or after transanal/rectal irrigation:

- Sustained or severe bleeding from the rectum
- Sustained or severe abdominal or back pain, with or without fever

Perforation of the bowel wall is extremely rare. However it is a serious and potentially fatal complication of transanal/rectal irrigation. Should this occur, immediate medical attention is required. The incidents should be reported to the competent authority in your country and/or to Qufora A/S at QA@qufora.com . Qufora A/S should also be notified in case of any other device malfunction.

Please observe

Transanal/rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum. Please contact your healthcare professional for further advice.

Recommended usage time

The water bag and click tube can be used up to 1 month (15 times for irrigation every second day and 30 times for irrigation every day). The pump and control unit can be used 90 times.

Should discolouration develop on any part of the system within the recommended usage time, the part should be replaced.

The Klick catheters are single-use only

If the single-use items of the product are reused, there is a risk of unwanted effects that can influence the function of the product and the health of the patient, the carers and third persons.

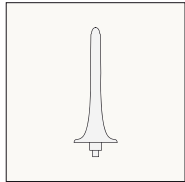
Please find further information about this product on www.qufora.com.

Distributor/UKRP:

Qufora Ltd,
Euro House,
Satellite Park, Macmerry,
East Lothian,
EH33 1RW

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