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EN	Instructions for Use
SV	Bruksanvisning
DE	Gebrauchsanweisung
DA	Brugsanvisning
NL	Gebruiksaanwijzing
FI	Käyttöohjeet
NO	Bruksanvisning
ES	Instrucciones de uso
FR	Mode d'emploi
IT	Istruzioni d'uso
HE	הוראות שימוש

3395-06_190325








www.qufora.com

Qufora A/S
Gydevang 28-30
3450 Allerød
Denmark

Tel: +45 88 300 592
info@qufora.com
www.qufora.com

Qufora® is a registered trademark of Qufora A/S. © 03/2025. All rights reserved Qufora A/S, 3450 Allerød, Denmark.

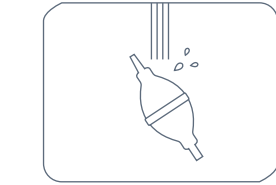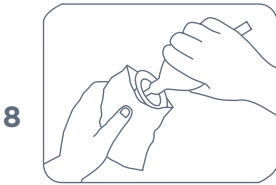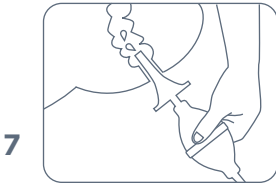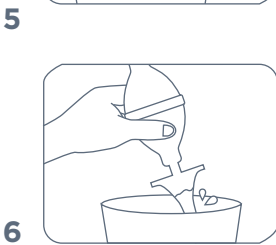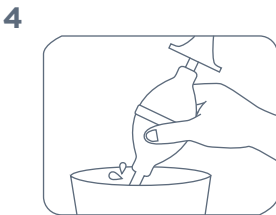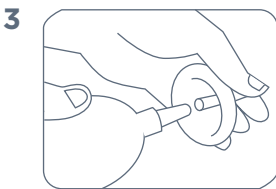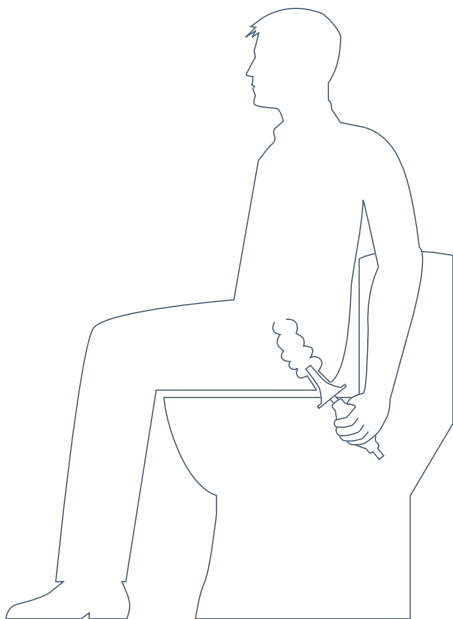English

Instructions for use

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Using the Qufora IrriSedo Mini makes it possible to perform transanal/rectal irrigation. Please read both pages of this instructions for use before using the Qufora IrriSedo Mini.

Indications for using transanal/rectal irrigation

The Qufora IrriSedo Mini is intended for adults and children 3 years and above to help manage faecal incontinence and/or constipation conditions.

For example:

- Passive faecal incontinence
- Post defaecation seepage
- Rectocele
- Incomplete evacuation
- Evacuation difficulties
- Low anterior resection syndrome

Within these groups some individuals may require the support of a carer to perform the procedure.

Use as advised by your healthcare professional

For the best results Qufora IrriSedo Mini should be used daily or as advised by your healthcare professional.

Before starting transanal/rectal irrigation

Use only after assessment and training from a qualified healthcare professional. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Transanal/rectal irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis
- Colorectal cancer/pelvic malignancy pre surgical removal
- Acute inflammatory bowel disease
- Acute diverticulitis
- Within 3 months of anal or colorectal surgery
- Within 4 weeks of endoscopic polypectomy
- Ischaemic colitis

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Please note:

- Transanal/rectal irrigation is not recommended for:
- Pregnancy or planned pregnancy or breastfeeding
 - Children under 3 years of age

Use transanal/rectal irrigation only after careful discussion with a relevant medical practitioner under the following circumstances (relative contraindications):

- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis)
- Active perianal sepsis (fistula or abscess, third or fourth degree haemorrhoids)
- Previous rectal or colonic surgery
- Diarrhoea of unknown aetiology
- Faecal impaction/rectal constipation
- Severe autonomic dysreflexia
- Severe diverticulosis or diverticular abscess
- Abdominal or pelvic irradiation
- Long term steroid therapy
- Anti-coagulant therapy
- Low blood sodium
- Previous severe pelvic surgery
- Colonic biopsy within the past 3 months
- Use of rectal medications for other diseases which may be diluted by irrigation
- Congestive cardiac failure

Adverse events and device malfunction

Seek medical attention if the following is experienced during or after transanal/rectal irrigation:

- Sustained or severe bleeding from the rectum
- Sustained or severe abdominal or back pain, with or without fever

Perforation of the bowel wall is extremely rare. However it is a serious and potentially fatal complication of transanal/rectal irrigation. Should this occur, immediate medical attention is required. The incidents should be reported to the competent authority in your country and/or to Qufora A/S at QA@qufora.com . Qufora A/S should also be notified in case of any other device malfunction.

Please observe

Transanal/rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum.

Please contact your healthcare professional for further advice.

Recommended usage time

The pump can be used up to 1 month. Should discolouration develop within the recommended usage time, pump should be replaced.

The cone is for single-use only

If the cone is reused, there is a risk of adverse effects to the product and the health of the patient, carers and third persons.

Please find further information about this product on www.qufora.com.

Distributor/UKRP:

Qufora Ltd,
Euro House,
Satellite Park, Macmerrey,
East Lothian,
EH33 1RW


Explaining the Qufora IrriSedo Mini components

Pump


A hand pump that connects to the cone (fig. 1).

Cone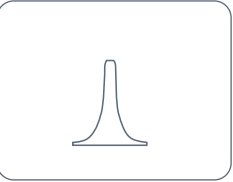

A soft cone coated with a hydrophilic lubricant that is activated by water (fig. 2).

Extension tube

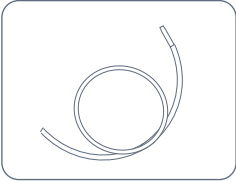As accessories a water bag to feed the pump and an extension tube to extend the distance between the pump and cone are available.

To use the system please follow the instructions:

1. Assemble the system by pushing the cone and the hand pump together (fig.3). The sticky label on the cone pouch can be attached to a surface within convenient reach. The cone pouch is ready to receive the cone after use.
2. Fill luke warm tap water (36-38°C) into a clean bowl or open container.
3. The hand pump is filled with water by compressing the pump, submerging the free end of the pump in the warm water and releasing the pump to draw the water into the pump (fig. 4).
4. To fill the pump completely, carefully compress the pump again until water seeps from the cone tip and release the pump with the free end in water.
5. To activate the lubricant simply bring the cone into contact with water (fig.5).
6. Seated on the toilet, carefully guide the cone into the rectum (fig. 6). While the cone is held in place gently compress the pump to pressure the water into the rectum.
7. Compress the pump completely just once to utilise as much water as possible. If the pump is compressed again, air will be pumped into the rectum.
8. Remove the pump. Faecal matter and water will then flow into the toilet.
9. The procedure can be repeated by repeating step 3– 8.
10. Place the soiled cone into the cone pouch (fig. 7). Hold the cone through the cone pouch to twist and pull the cone off the pump. Fold the open end of the cone pouch twice and use the sticky label to close it. Throw it away in your household waste.
11. If required, all external surfaces of the system can be cleaned with lukewarm water and mild soap (fig 8). Dry with a lint free cloth.

Please note: Please use Qufora IrriSedo Mini only as advised by your healthcare professional.

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