Qufora® IrriSedo Bed

- EN** Qufora® IrriSedo Bed **Instructions for Use**
- SV** Qufora® IrriSedo Bed **Bruksanvisning**
- DE** Qufora® IrriSedo Bed **Gebrauchsanweisung**
- DA** Qufora® IrriSedo Bed **Brugsanvisning**
- NL** Qufora® IrriSedo Bed **Gebruiksaanwijzing**
- FI** Qufora® IrriSedo Bed **Käyttöohjeet**
- NO** Qufora® IrriSedo Bed **Bruksanvisning**
- ES** Qufora® IrriSedo Bed **Instrucciones de uso**
- EL** Qufora® IrriSedo σύστημα υποκλιτισμού στο κρεβάτι **Οδηγίες χρήσης**
- FR** Qufora® IrriSedo Bed **Mode d'emploi**
- IT** Qufora® IrriSedo Bed **Istruzioni d'uso**
- HE** Qufora® IrriSedo **הוראות שימוש למערכת למיטה**

011220-3397-03

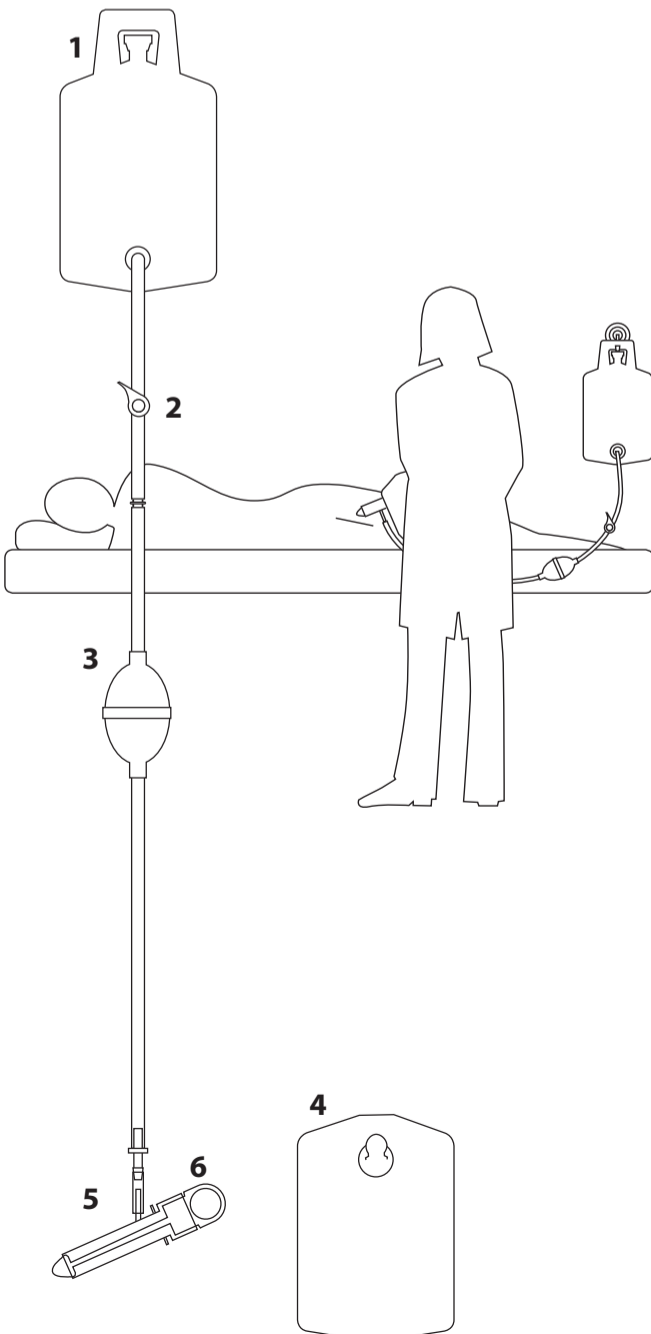
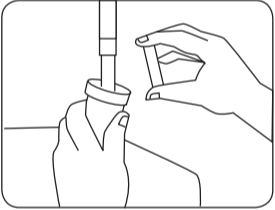
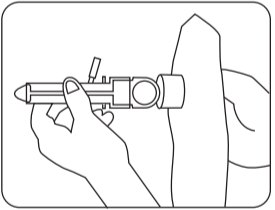
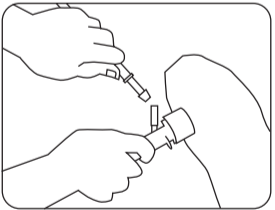
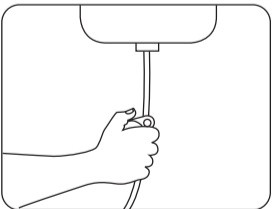
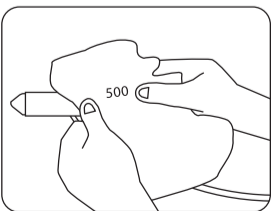
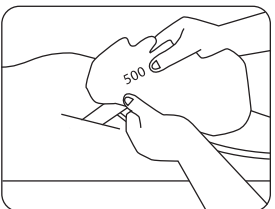
Qufora A/S
 Gydevang 28-30
 3450 Allerød
 Denmark

Tel: +45 88 300 592
 Fax: +45 88 300 593
 info@qufora.com
 www.qufora.com

Qufora® is a registered trademark of Qufora A/S. All rights reserved Qufora A/S, 3450 Allerød, Denmark.

- 7**  Illustration showing hands being washed with soap and water.
- 8**  Illustration showing the nozzle being prepared for use.
- 9**  Illustration showing the nozzle being connected to the tube.
- 10**  Illustration showing the pump being adjusted.
- 11**  Illustration showing a '500' mark being made on the patient's skin.
- 12**  Illustration showing the final marking on the patient's skin.

English

Qufora® IrriSedo Irrigation System

Using the Qufora® IrriSedo Irrigation System makes it possible to perform rectal irrigation. Please read both pages of this instruction before using the Qufora® IrriSedo Irrigation System.

Application

The Qufora® IrriSedo Irrigation System is used to prevent chronic faecal incontinence and/or constipation or for those who spend a long time on bowel management. It should be used regularly - usually daily or every other day to provide the best effect.

Before starting rectal irrigation

Use only after assessment, approval and full instruction by a doctor/healthcare professional who is specialised in this procedure. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Indications for using rectal irrigation

Individuals with the following conditions and symptoms may benefit:

- Neurogenic bowel dysfunction, e.g. spinal cord injury, spina bifida and multiple sclerosis.
- Chronic constipation.
- Chronic faecal incontinence.

The Qufora® IrriSedo bed system requires the assistance of a carer to perform the procedure.

Irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis.
- Colorectal cancer pre surgical removal.
- Acute inflammatory bowel disease.
- Acute diverticulitis.
- Within 3 months of anal or colorectal surgery.
- Within 4 weeks of endoscopic polypectomy.
- Ischaemic colitis.

Since the list is not exhaustive a doctor/healthcare professional should always consider individual patient factors as well.

Use rectal irrigation only after careful discussion with relevant medical practitioner under the following circumstances (relative contraindications):

- Pregnancy or planned pregnancy or breast feeding.
- Children under 3 years of age, considering the size and shape of the device

delivering the water. For children the approval should be from a qualified medical practitioner specialised in paediatric care.

- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis).
- Active perianal sepsis (fistula or abscess).
- Rectal or colonic surgery within the past 6 months.
- Diarrhoea of unknown aetiology.
- Faecal impaction/ rectal constipation.
- Severe autonomic dysreflexia.
- Severe diverticulosis or diverticular abscess.
- Abdominal or pelvic radiotherapy.
- Known diverticular disease.
- Long term steroid therapy.
- Anti-coagulant therapy.
- Low blood sodium.
- Pelvic malignancy.
- Colonic biopsy within the past 3 months.
- Use of rectal medications for other diseases which may be diluted by irrigation.
- Congestive cardiac failure.

Warning

Perforation of the bowel wall is extremely rare. However it is a serious and potentially lethal complication of rectal irrigation. Should this occur, immediate medical attention is required.

Seek medical attention if the following is experienced during or after rectal irrigation:

- Sustained or severe bleeding from the rectum.
- Sustained or severe abdominal or back pain, with or without fever.

If the single use items of the product are reused, there is a risk of unwanted effects that can influence the function of the product and the health of the patient, the caregivers and third persons.

Please observe

Rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum.

By following the recommended water volumes and water temperatures the bowel electrolyte balance is unlikely to be affected.

Recommended usage time

(when used routinely every other day): The catheter and collection bag are single use items.

The water bag and pump can be used up to 1 month. Should discolouration develop within the recommended usage time, this item should be replaced.

Liability

MBH-International A/S accepts no liability for any injury or other loss that may arise if this product is used in any way contrary to the recommendations of MBH-International A/S.

Handling and storage

Qufora® IrriSedo Irrigation System should be stored at temperatures between 0 and 30° C.

Protect the product from direct sunlight and do not expose to heavy pressure.

The product should be kept out of reach of children.

Further information about this product can be found on www.qufora.com.

Distributor United Kingdom

MacGregor Healthcare Ltd
Unit 2A, Euro House
Satellite Park, Macmerry
East Lothian
EH33 1RW

Distributor Ireland

Uniphar Medical,
4045 Kingswood Road,
Citywest Business Park,
Co. Dublin
D24 V06K

How to use the system

The system comprises:

- (fig. 1) A water bag fitted with a valve (fig. 2) and a pump (fig. 3).
- (fig. 4) A collection bag that holds 3.0 litres.
- (fig. 5) A rectal catheter with an insertion guide (fig. 6) (for Ø21 and Ø30 catheters only).

Use irrigation only after assessment and approval by a qualified health care professional.

To use the system please follow the instructions:

- 1 Fill the water bag with warm (36–38° C) tap water (fig. 7) (only fill the bag up with the amount of water you need for your irrigation). Place the bag no higher than 50 cm above the level of the mattress. The length of the tube from water bag to the valve is 50 cm.
 - 2 Fit the collection bag to the catheter with the insertion guide in the catheter (fig. 8). Note that the size Ø12 mm catheter does not require a guide.
 - 3 Connect the tube to the rectal catheter (fig. 9). Open the valve to fill the tube and pump with water (fig.10). Close the valve again. Lubricate the tip of the catheter guide with a water based lubricant.
 - 4 Hold the ring of the insertion guide with the scale of the collection bag pulled up between the 500 and 1000 ml. marking (fig.11).
 - 5 Position the patient on their left side. Carefully insert the rectal catheter into the rectum with the insertion guide (fig.12).
 - 6 With the catheter in the rectum pull out the insertion guide and let it drop into the collection bag.
 - 7 Push the plug in the collection bag into the open end of the catheter.
 - 8 Support the rectal catheter to avoid it being pushed out of the rectum. Open the valve on the tube. Start by opening the valve half way. If the water bag is placed at the mattress level or lower, open the valve completely and pump water into the rectum.
 - 9 Allow approximately 300 – 500 ml.(less for children – as specified by the healthcare professional) of water to enter the rectum and close the valve. **Use max. 1000 ml. per irrigation.** If any discomfort is felt, close the valve immediately.
- Please note: Should water seep out along the catheter the cause may be:**
- Too much water has entered the rectum.
 - The muscle tone of the sphincter is too low to hold the water.
 - The diameter of the catheter used is too small (The catheter is available in the diameters Ø12, Ø21 and Ø30 mm).
- 10 When the water is instilled, pull the plug out of the rectal catheter, water and faecal matter will flow into the collection bag.
 - 11 The procedure can be repeated by repeating step 7 to 10.
 - 12 Carefully remove the rectal catheter. Pull the water bag tube off the catheter. Empty the contents of the collection bag and dispose of the catheter and collection bag by placing them into a disposal bag before putting it into clinical waste. Wash the water bag and pump with mild soap. Hang the parts to dry with the lid off the bag. Wash your hands to complete the procedure.

To gain faecal continence or to resolve constipation, irrigation should be used routinely daily or every other day.

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