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
Tel: +45 88 300 592
Fax: +45 88 300 593
info@qufora.com
www.qufora.com

 Qufora®


Qufora® is a registered trademark of Qufora A/S. All rights reserved Qufora A/S, 3450 Allerød, Denmark.

1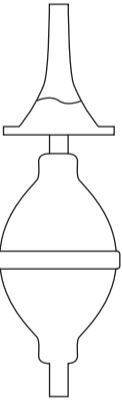


2


3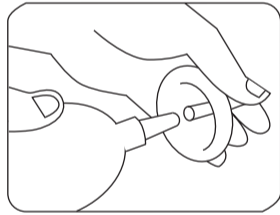


4

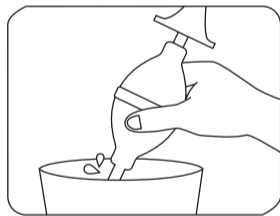
5

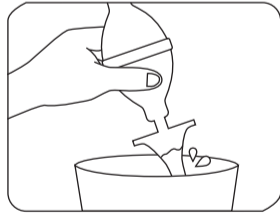
6

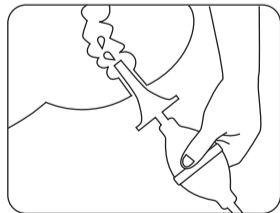7

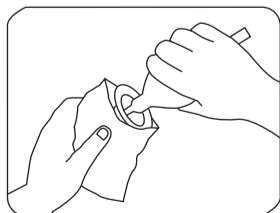8

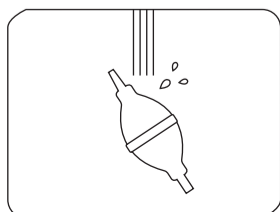English

Qufora® IrriSedo Irrigation System

Using the Qufora® IrriSedo Irrigation System makes it possible to perform rectal irrigation. Please read both pages of this instruction before using the Qufora® IrriSedo Irrigation System.

Application

The Qufora® IrriSedo Irrigation System is used to prevent chronic faecal incontinence and/or constipation or for those who spend a long time on bowel management. It should be used regularly - usually daily or every other day to provide the best effect.

Before starting rectal irrigation

Use only after assessment, approval and full instruction by a healthcare professional who is specialised in this procedure. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Indications for using rectal irrigation

Individuals with the following conditions and symptoms may benefit:

- Neurogenic bowel dysfunction, e.g. spinal cord injury, spina bifida and multiple sclerosis.
- Chronic constipation.
- Chronic faecal incontinence.

Within these groups some individuals may require the support of a carer to perform the procedure.

Irrigation should not be used under the following circumstances (absolute contraindications):

- Known anal or colorectal stenosis.
- Colorectal cancer pre surgical removal.
- Acute inflammatory bowel disease.
- Acute diverticulitis.
- Within 3 months of anal or colorectal surgery.
- Within 4 weeks of endoscopic polypectomy.
- Ischaemic colitis.

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Use rectal irrigation only after careful discussion with relevant medical practitioner under the following circumstances (relative contraindications):

- Pregnancy or planned pregnancy or breast feeding.
- Children under 3 years of age, consid-

ering the size and shape of the device delivering the water. For children the approval should be from a healthcare professional specialised in paediatric care.

- Inflammatory bowel disease (e.g. Crohn's disease or ulcerative colitis).
- Active perianal sepsis (fistula or abscess).
- Rectal or colonic surgery within the past 6 months.
- Diarrhoea of unknown aetiology.
- Faecal impaction/ rectal constipation.
- Severe autonomic dysreflexia.
- Severe diverticulosis or diverticular abscess.
- Abdominal or pelvic radiotherapy.
- Known diverticular disease.
- Long term steroid therapy.
- Anti-coagulant therapy.
- Low blood sodium.
- Pelvic malignancy.
- Colonic biopsy within the past 3 months.
- Use of rectal medications for other diseases which may be diluted by irrigation.
- Congestive cardiac failure.

Warning

Perforation of the bowel wall is extremely rare. However it is a serious and potentially lethal complication of rectal irrigation. Should this occur, immediate medical attention is required.

Seek medical attention if the following is experienced during or after rectal irrigation:

- Sustained or severe bleeding from the rectum.
- Sustained or severe abdominal or back pain, with or without fever.

If the single use items of the product are reused, there is a risk of unwanted effects that can influence the function of the product and the health of the patient, the carers and third persons.

Please observe

Rectal irrigation can be associated with passing sensations of bowel discomfort, nausea, shivering, fatigue, sweating, headache and light bleeding from the rectum.

By following the recommended water volumes and water temperatures the bowel electrolyte balance is unlikely to be affected.

Recommended usage time (when used routinely every other day):

The pump can be used up to 1 month. Should discolouration develop within

the recommended usage time, pump should be replaced.

The cone is a single use item.

Liability

MBH-International A/S accepts no liability for any injury or other loss that may arise if this product is used in any way contrary to the recommendations of MBH-International A/S.

Handling and storage

Qufora® IrriSedo Irrigation System should be stored at temperatures between 0 and 30° C.

Protect the product from direct sunlight and do not expose to heavy pressure.

The product should be kept out of reach of children.

Further information about this product can be found on www.qufora.com.

Distributor United Kingdom

MacGregor Healthcare Ltd
Unit 2A, Euro House
Satellite Park, Macmerry
East Lothian
EH33 1RW

Distributor Ireland

Unipharm Medical,
4045 Kingswood Road,
Citywest Business Park,
Co. Dublin
D24 V06K

How to use the system

The system comprises:

- (fig.1) A soft rectal cone coated with a hydrophilic lubricant that is activated by water.
- (fig.2) A hand pump that connects to the cone.

The hand pump contains approximately 90 ml. of water.

Note: The hand pump has a set of one way valves so water can only enter the pump from the bottom and flow out from the top.

As accessories a water bag to feed the pump and an extension tube to extend the distance between the pump and cone are available.

Use irrigation only after assessment and approval by a qualified healthcare professional.

To use the system, follow the instructions:

- 1 Assemble the system by pushing the cone and the hand pump together (fig.3). The sticky label on the cone pouch can be attached to a surface within convenient reach. The cone pouch is ready to receive the cone after use.
- 2 Fill clean water (36 – 38 °C) into a clean bowl or open container.
- 3 The hand pump is filled with water by compressing the pump, submerging the free end of the pump in the warm water and releasing the pump to draw the water into the pump (fig. 4).
- 4 To fill the pump completely, carefully compress the pump again until water seeps from the cone tip and release the pump with the free end in water.
- 5 To activate the lubricant simply bring the cone into contact with water (fig.5).
- 6 Seated on the toilet, carefully guide the cone into the rectum (fig. 6). While the cone is held in place gently compress the pump to pressure the water into the rectum.
- 7 Compress the pump completely **just once** to utilise as much water as possible. If the pump is compressed again, air will be pumped into the rectum.
- 8 Remove the pump. Faecal matter and water will then flow into the toilet.
- 9 The procedure can be repeated by repeating step 3– 8.
- 10 Place the soiled cone into the cone pouch (fig. 7). Hold the cone through the cone pouch to twist and pull the cone off the pump. Fold the open end of the cone pouch twice and use the sticky label to close it. Throw it away in your household waste.
- 11 Clean the pump with mild soap (fig. 8). Wash your hands to complete the procedure.

To gain faecal continence or to resolve constipation, irrigation should be used routinely daily or every other day.

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Brainstorm Marketing Ltd